



IMPORTANT START UP INFORMATION

Verify the LifeChoice is fully powered **ON** before placing cannula in nasal passages. This is indicated by the green LED lights next to the Active button and setting number 1 button. These lights should be on solid green. Breathing through the cannula when starting (powering on) the LifeChoice will cause the system to immediately power off. The LifeChoice is recommended and designed to be operated and stored in its carry case at all times.

IMPORTANT OPERATING INFORMATION

Before using the LifeChoice, please read the following information:

- 1) The LifeChoice and External Battery pack should be charged for a minimum 6 hours before initial use. Connect the External Battery pack to the LifeChoice, connect the AC adapter to the External Battery pack and plug AC adapter into electrical outlet.
- 2) Battery Charging times:
Internal Battery Charge Time: 4 hours
External Battery Charge Time: 6 hours
Both batteries simultaneously: 6 hours
- 3) For optimum performance the LifeChoice should be powered using the external battery first and then the internal battery. The LifeChoice will automatically switch to the internal battery once the external battery has been depleted.
- 4) When powered by the internal battery, the green LED Battery Check light on the top of the LifeChoice flashes. When powered by an external source i.e. AC/DC Adapter or External Battery pack, the Battery Check LED is a solid green.
- 5) Whenever an external power source is plugged into the LifeChoice, make sure you verify the green LED Battery Check light is solid. This indicates the system recognizes the external power source.
- 6) If the External battery is plugged into the LifeChoice and the green LED Battery Check light does not turn solid, the source is not recognized. If charging the unit through the External battery when this occurs, the charge will not reach the internal battery. This may be an indication the External Battery needs to be reset. Instructions for resetting the external battery are in step 7 below.
- 7) Resetting the External Battery:
Remove the battery from the case. On the same end as the battery charging cord, is a small pink reset button and a LED light. If the LED has turned red, the battery pack needs to be reset. Press and hold the reset button until the red LED shuts off. Return battery to case and connect to LifeChoice. Verify the green LED Battery Check light on the LifeChoice is solid green. If the light has not turned to a solid green after this procedure, contact your provider for further support.

INOVA LABS • 2101 E. ST. ELMO RD SUITE 275 • AUSTIN, TEXAS 78744 •
512-326-3237 • FAX 512-326-3299
www.lifechoiceoxygen.com
1-877-490-3339 Toll Free 24/7 Call Center